# Fußpflege | Podologie | Nail

*Chiropody | podiatry | nail accessories*

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*Some of the products and designations mentioned in the text are patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of copyright protection.*

*This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or processed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.*

*Colours and products subject to alterations. Printing errors excepted.*

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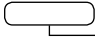
### *Welcome to Komet Medical*

*Every day, almost 1100 dedicated employees at Komet use their vast expertise to meet the requirements of modern medicine. As the leading German manufacturer of rotary instruments worldwide, we offer our clients exceptional quality based on the careful choice of the raw material to the benefit of the final user. Since 1923, the brand Komet has been an internationally renowned synonym for utmost precision and absolute reliability.*

*To meet the ever growing expectations of our users and to continue supplying products of superb quality, we not only constantly invest in the latest technologies, innovative materials and highly qualified staff, but also prove our competence by conquering new business domains. In the year 2010, we expanded our medical sector Komet Medical by adding a chiropody/podiatry/nail care section. We are now proud to offer a complete range of reliable, high-quality rotary instruments for the successful treatment of patients in this sector.*

*Best regards  
Your Komet Medical team*

		<b>Hartmetallfräser</b> <i>Tungsten carbide cutters</i>	<b>14 - 23</b>
		<b>RF-Stahlfräser</b> <i>Stainless steel cutters</i>	<b>25 - 27</b>
		<b>Polierer &amp; Bürsten</b> <i>Polishers &amp; brushes</i>	<b>30 - 31</b>
		<b>Instrumentensets</b> <i>Instrument sets</i>	<b>35</b>
<b>Allgemeine Gebrauchs- und Sicherheitshinweise</b> <i>General instructions for use and safety recommendations</i>		<b>38 - 39</b>	
<b>Notizen</b> <i>Notes</i>		<b>42 - 43</b>	
		<b>Diamantschleifer</b> <i>Diamond abrasives</i>	<b>6 - 13</b>
		<b>Keramikfräser</b> <i>Ceramic cutters</i>	<b>24</b>
		<b>Turbineninstrumente</b> <i>Turbine instruments</i>	<b>28 - 29</b>
		<b>NailDesign</b>	<b>32 - 34</b>
		<b>Zubehör</b> <i>Accessories</i>	<b>36 - 37</b>
		<b>Anwendungskompass mit Drehzahlempfehlungen</b> <i>User compass with recommended speed</i>	<b>40 - 41</b>

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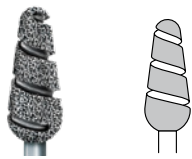
#### Extra coarse grit

Removal of very hard skin and calluses

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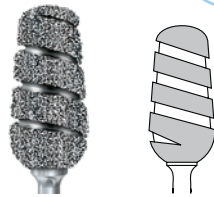
### Callus sander AXC

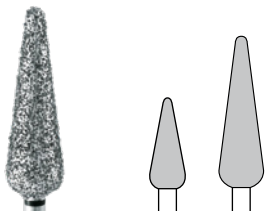
- Newly developed instrument with spiral groove
- Guarantees excellent substance removal
- Reduced heat generation thanks to the spiral-shaped interruption of the surface,
- ... which makes it equally suitable for dry techniques
- Optimum use with spray in combination with equipment using wet techniques

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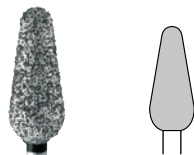
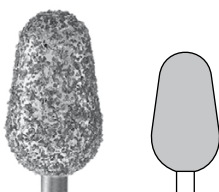
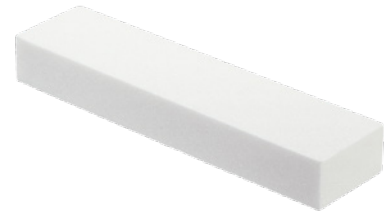
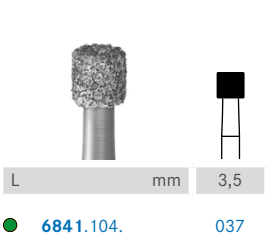
### Coarse grit

For removing calluses and treatment of nails

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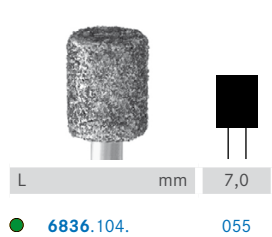
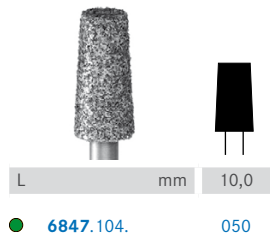
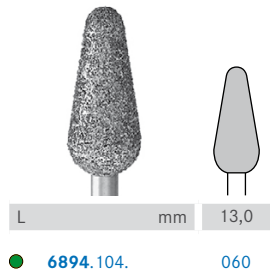
#### KR – rounded edges

- Smoothing of hard skin and nails
- Minimum risk of injury
- More safety for at-risk patients

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#### T – diamond coated on Top only

- Ideal for shortening nails
- The adjacent skin remains untouched
- Safety bevel at the edge of the front end

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#### S – diamond coated on the periphery only

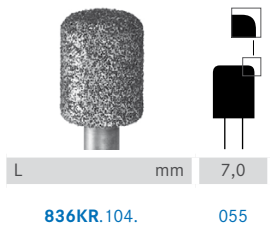
- Safe abrasion up to the edge of the nail
- Greatly reduced risk of injury
- Safety bevel at the edge of the front end

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

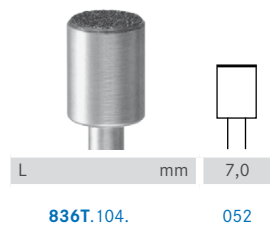
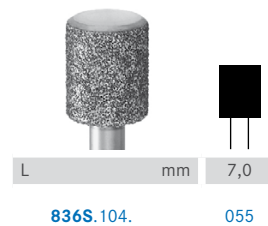
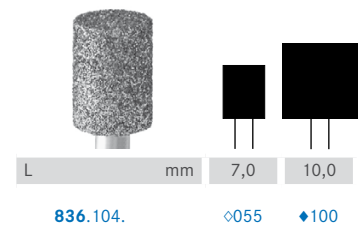
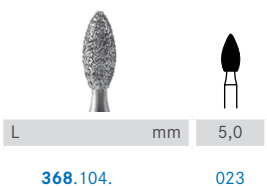
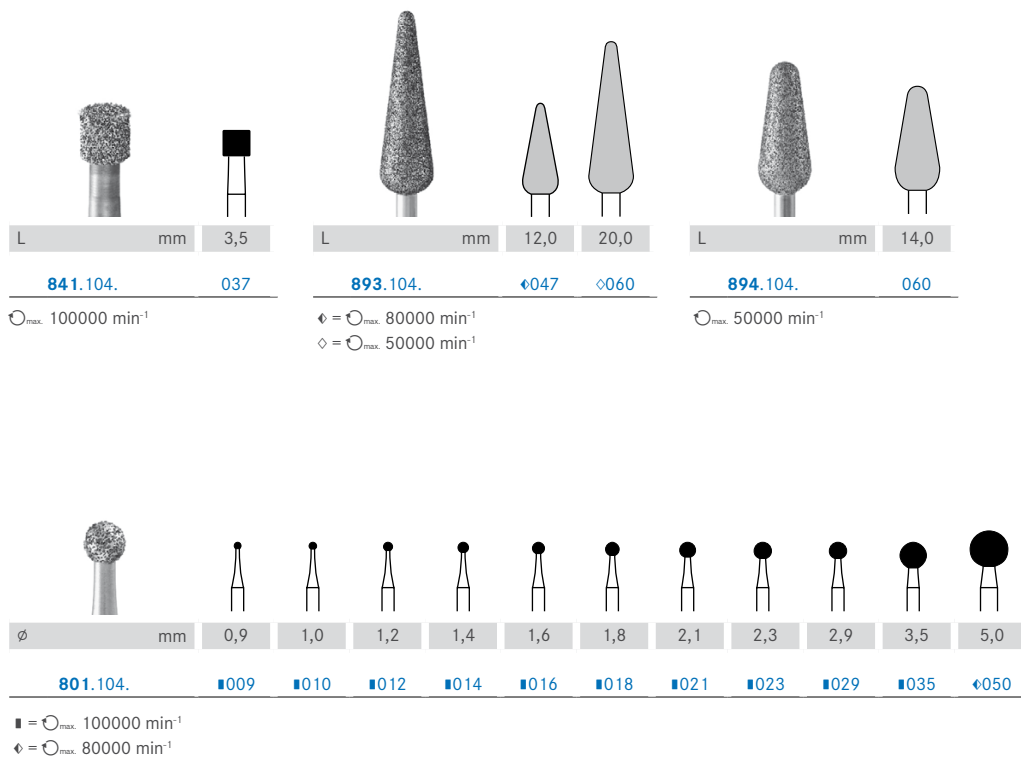
#### *Our diamond instruments are of outstanding quality*

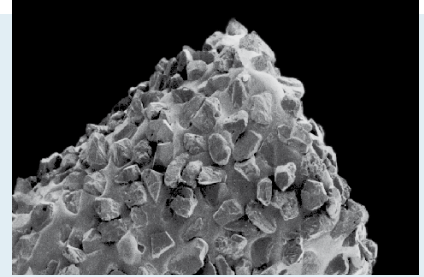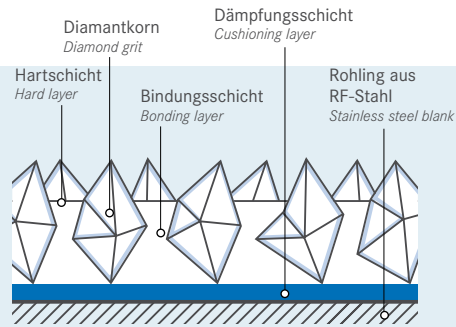*Our diamond instruments are produced according to a sophisticated procedure which guarantees an optimum structure. We only use high-quality diamond grit. The grit size is perfectly adapted to the intended use.*

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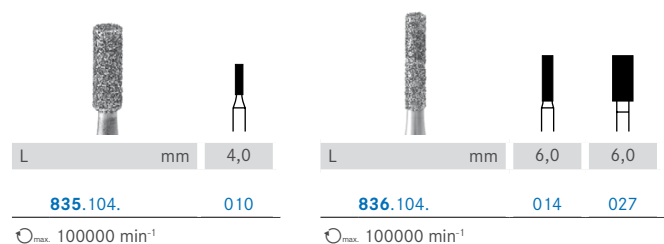
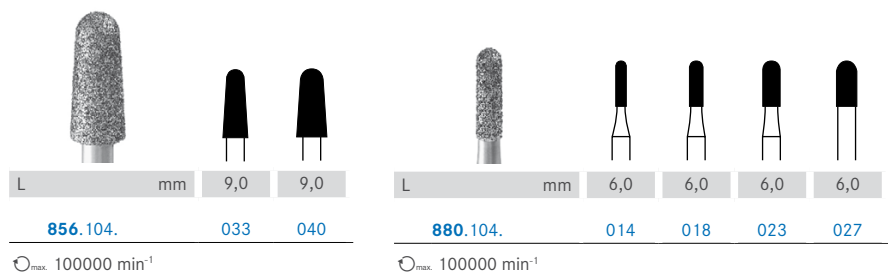
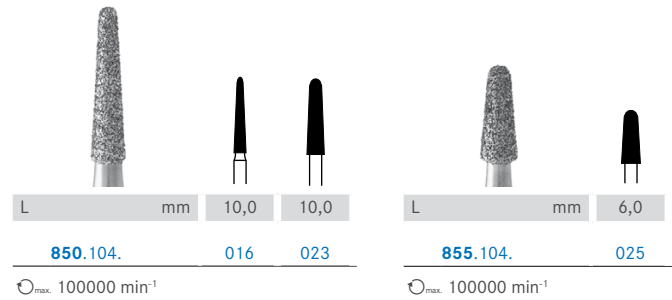
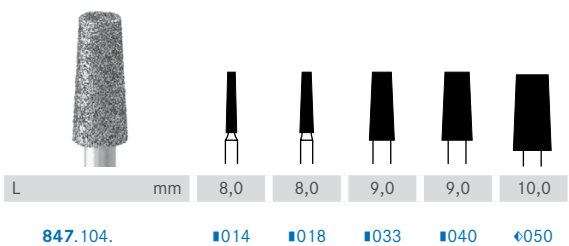
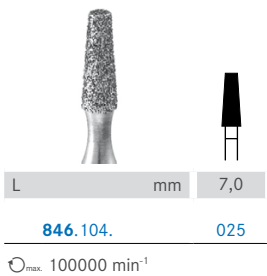
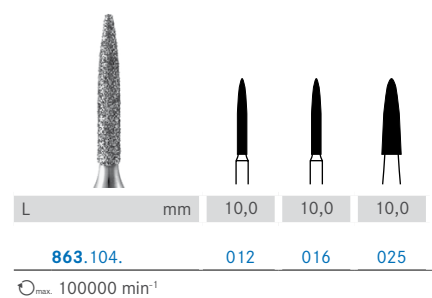
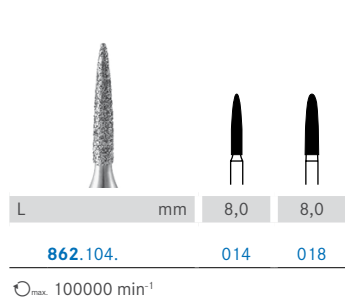
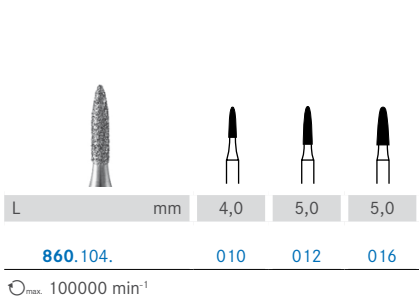
#### Recommendations for cleaning diamond abrasives

Heavily used instruments require special care to guarantee proper cleaning and disinfection and to avoid a reduction of the cutting performance.

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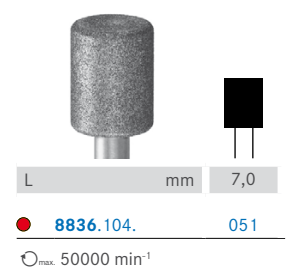
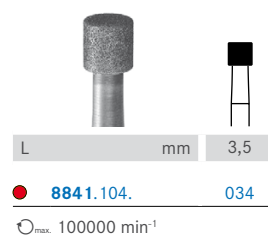
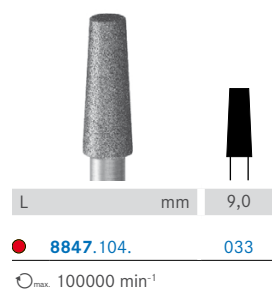
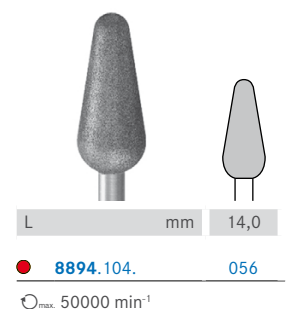
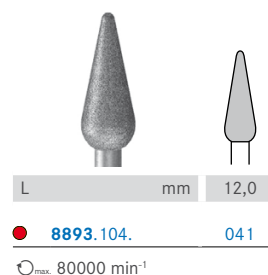
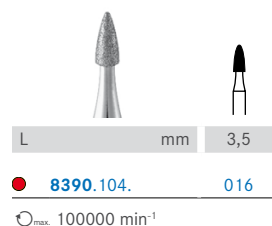
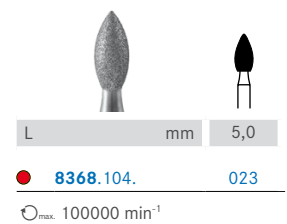
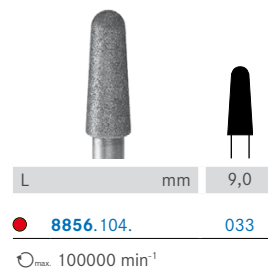
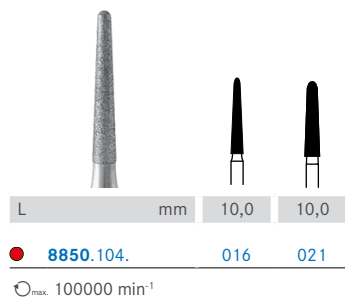
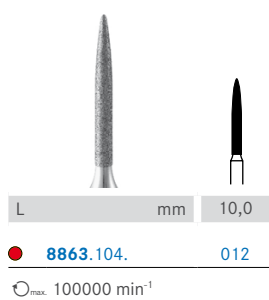
- *Do not use cleaning brushes made of brass wire. These are unsuitable for diamond coated instruments.*

- *The specially designed cleaning brush 9873 with fine nylon bristles is ideally suitable for cleaning diamond abrasives.*

- *Cleaning stone 9750 for pre-cleaning heavily contaminated diamond abrasives. Suitable only for use in wet technique at low speed.*

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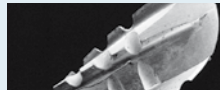
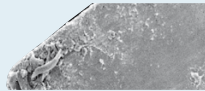
**Not all types of tungsten carbide are the same**

The tungsten carbide instruments made by Komet Medical meet the highest expectations, thanks to:

- Hot isostatically pressed tungsten carbide, calibrated to 1 µ and compressed with a pressure of 2.7 bar
- Sharp blades without nicks for efficient, gentle and precise incisions

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*H79GP – Treatment of mycotic nails without compromise*

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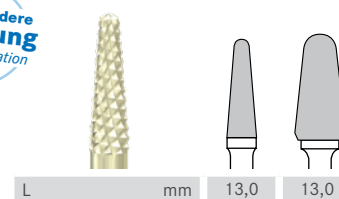
#### GP and P cutters

- Newly developed instruments with coarse and medium diamond toothing
- Suitable for use in dry technique at low to medium speed (15.000 – 20.000 rpm)
- Less heat generation thanks to improved surface cooling
- Reduced clogging of the chip spaces

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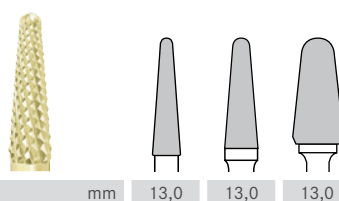
*Made of anodized aluminium, suitable for sterilisation*

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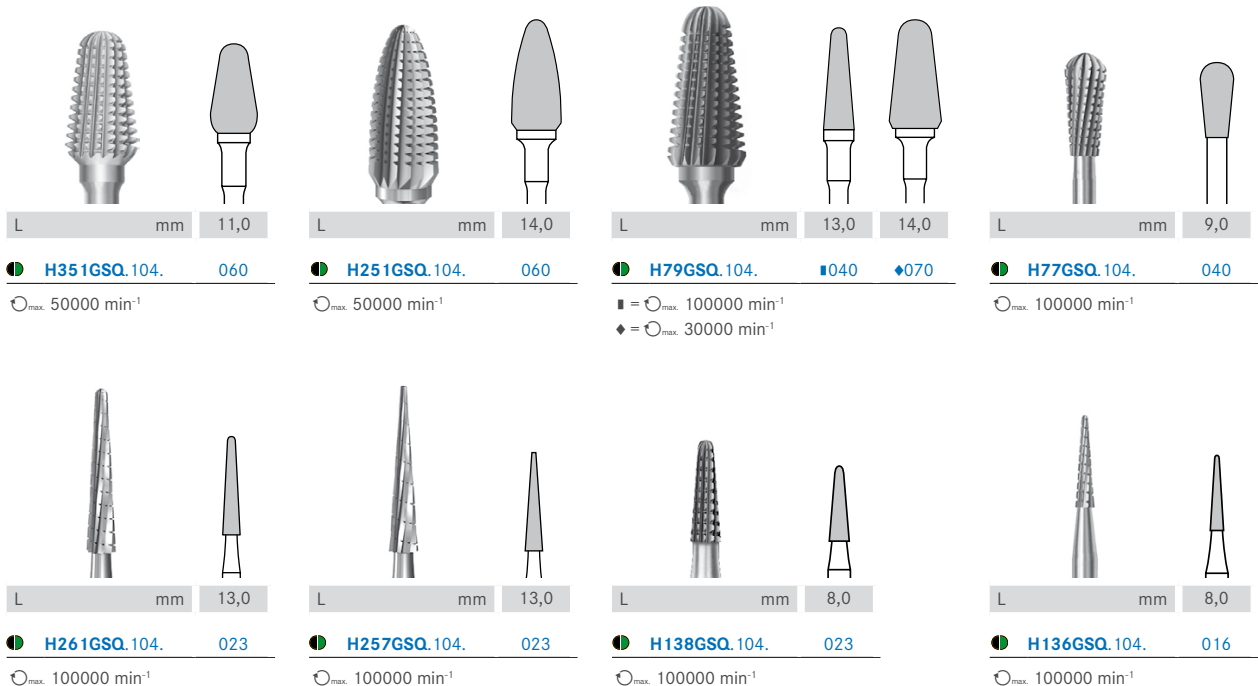
### GSQ cutters

- **Coarse** cross-cut toothing with effective cutting performance
- Effective cutting without clogging
- Excellent substance removal
- Gentle or strong milling, depending on the contact pressure applied

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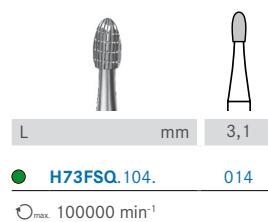
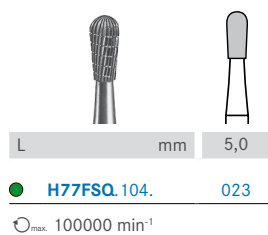
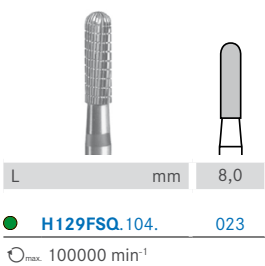
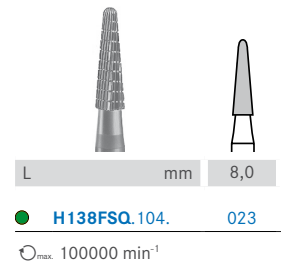
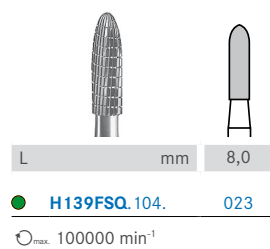
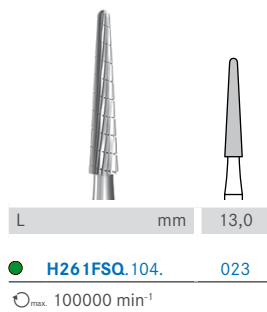
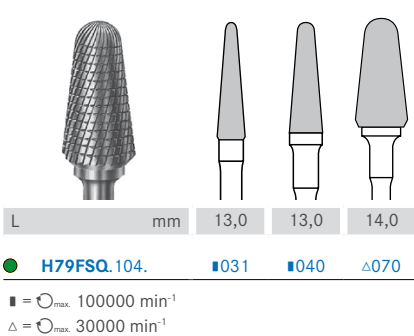
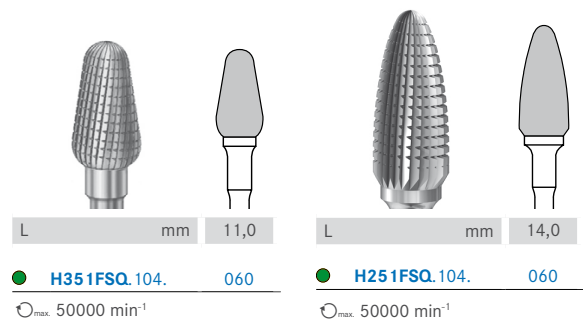
- **Fine** cross-cut tothing with effective cutting performance
- Suitable for final smoothing of nails and skin and for shaping of orthotic acrylics
- Effective cutting without clogging
- Particularly gentle milling due to low contact pressure
- Low heat generation thanks to small cutting segments
- Optimum surface quality

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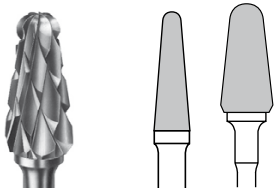
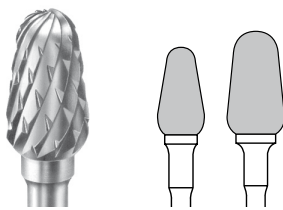
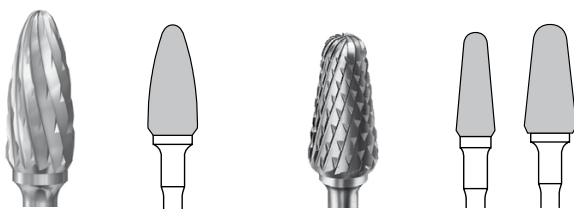
*Contrary to popular opinion, the efficiency of a bur is not in proportion to its speed. In other words, a higher speed does not necessarily equal greater substance removal. Quite the contrary, once a certain speed has been reached, the opposite is the case, i.e. there will be a certain loss of efficiency. What's more, the instrument heats up excessively which leads to premature wear.*


⚠ The optimum speed of cutters with a head diameter > 2,3 mm is between 10.000 – 20.000 rpm

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
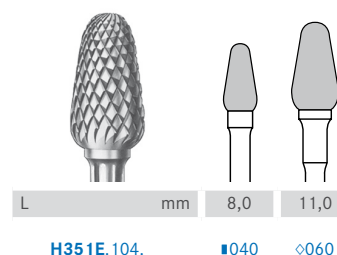
- Medium staggered toothing
- For universal treatments and the shaping of nails
- Suitable for treating fungal infections of the nail (mycosis)
- Low vibration for improved ergonomics whilst working

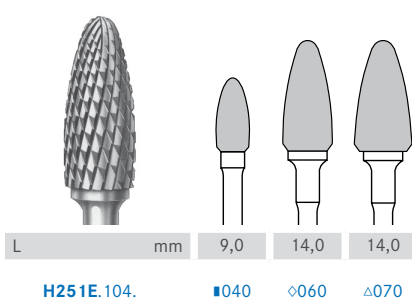
■ =  $\odot_{\max}$  100000 min<sup>-1</sup>  
 ◇ =  $\odot_{\max}$  50000 min<sup>-1</sup>  
 △ =  $\odot_{\max}$  30000 min<sup>-1</sup>

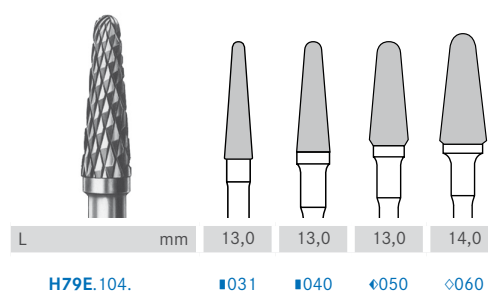
■ =  $\odot_{\max}$  100000 min<sup>-1</sup>  
 ◇ =  $\odot_{\max}$  50000 min<sup>-1</sup>  
 △ =  $\odot_{\max}$  30000 min<sup>-1</sup>

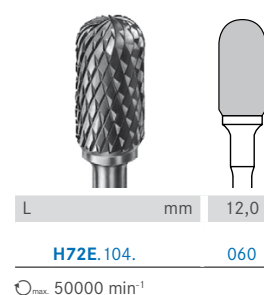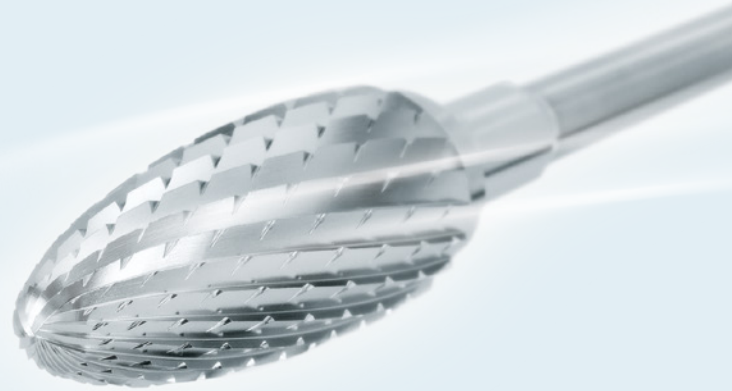■ =  $\odot_{\max}$  100000 min<sup>-1</sup>  
 ◇ =  $\odot_{\max}$  80000 min<sup>-1</sup>  
 ◇ =  $\odot_{\max}$  50000 min<sup>-1</sup>

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




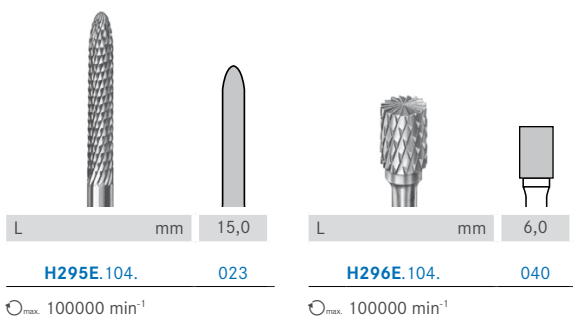
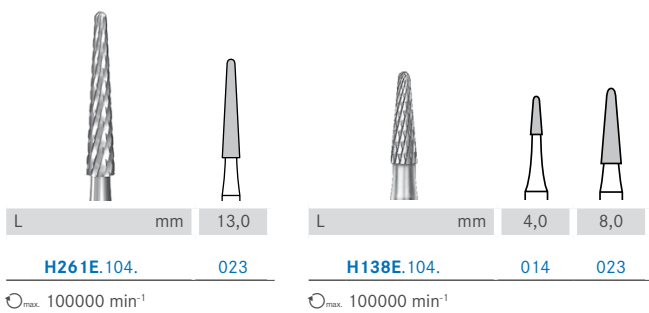
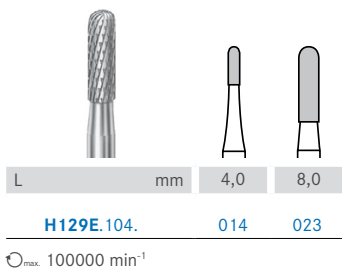
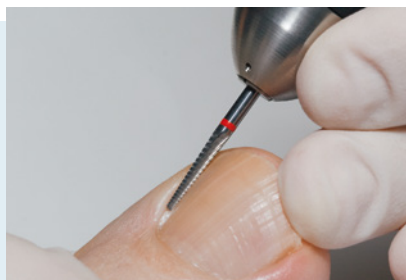
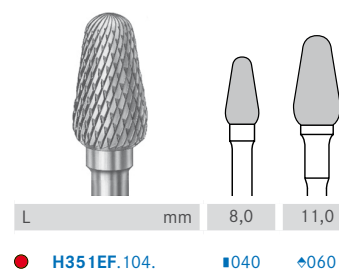
- *Fine staggered tothing*
- *For the final treatment of nail surfaces*
- *Ideally suited for smoothing the nail to facilitate subsequent polishing*
- *Low vibration for improved ergonomics whilst working*
- *Gentle milling with low contact pressure*

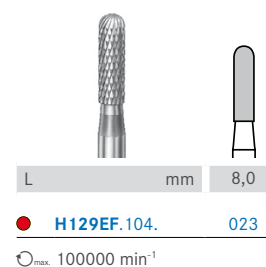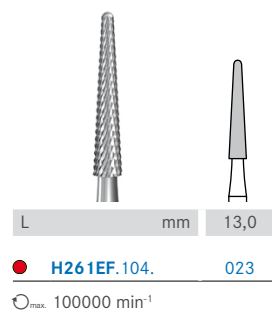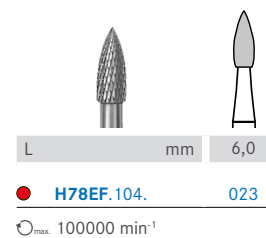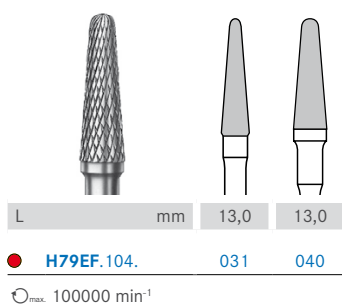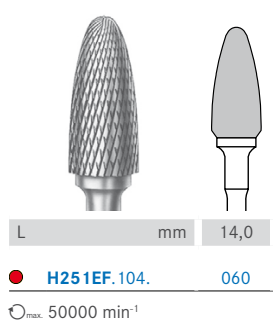■ =  $\odot_{\max}$  100000 min<sup>-1</sup>  
◆ =  $\odot_{\max}$  60000 min<sup>-1</sup>

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
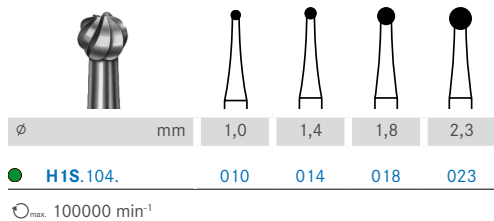
*Removal of corns, treatment of fungal infections of the nail (mycosis)*

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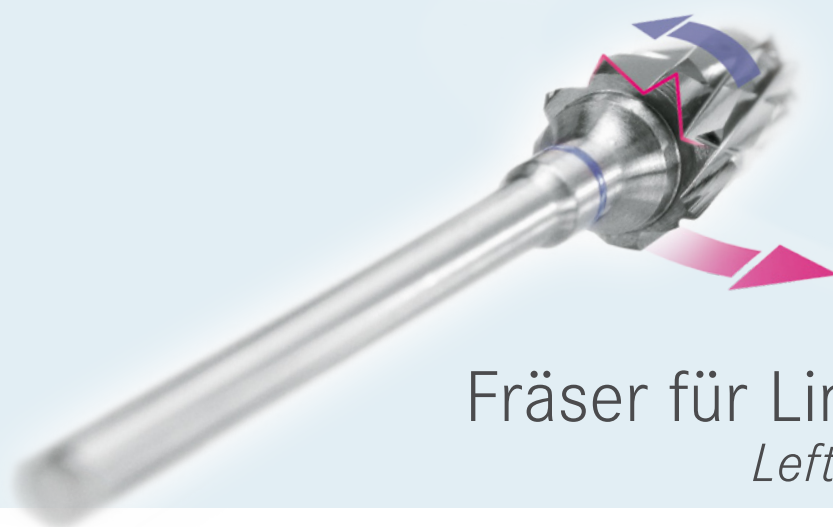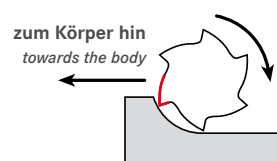
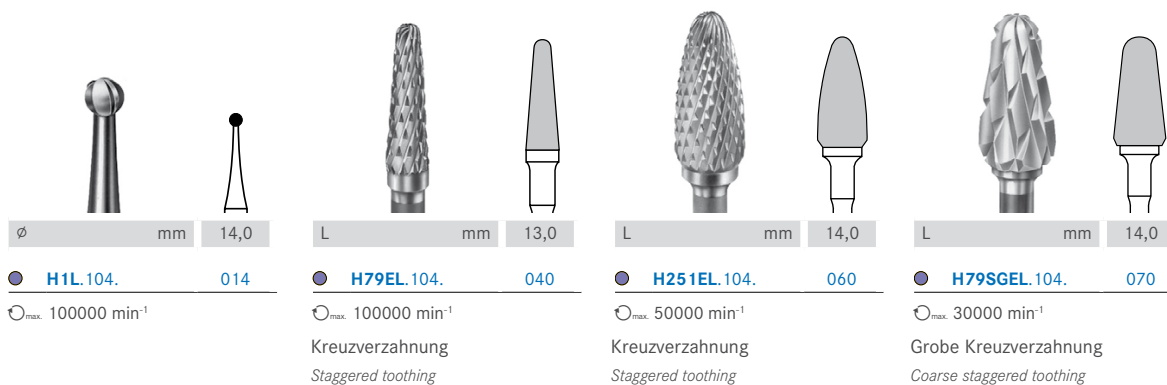
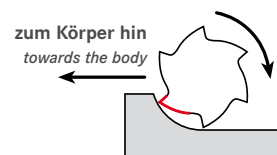
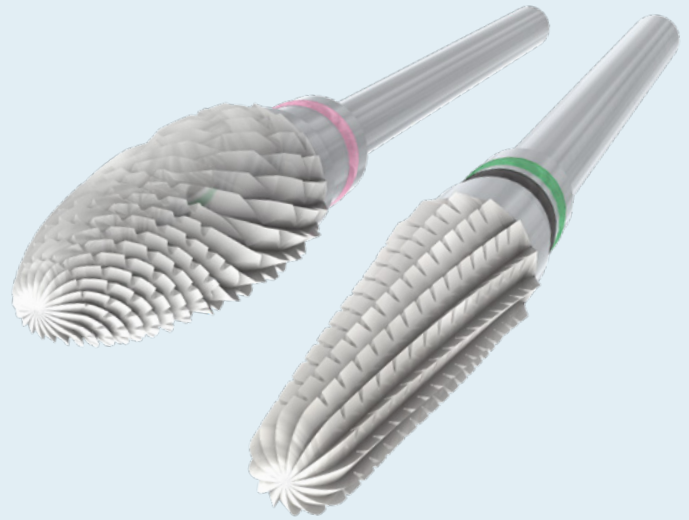
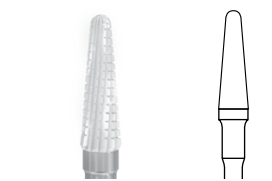
### Ceramic cutter

Treatment of the nail plate, fissures and hard skin

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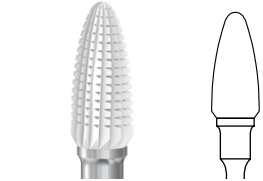
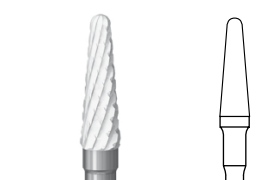
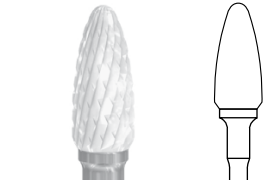
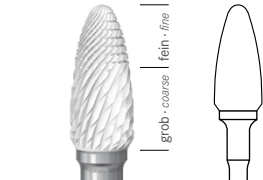
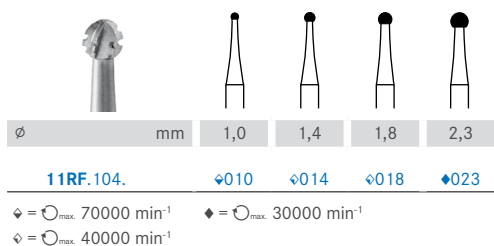
#### Round cutter

*With cross-cut, for removing corns and  
calluses from the skin*

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
#### Round cutter

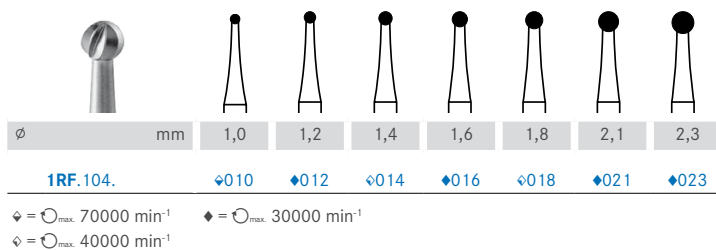*Without cross-cut, for removing corns and  
fungal infections from the nail*

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
#### Fissure cutter

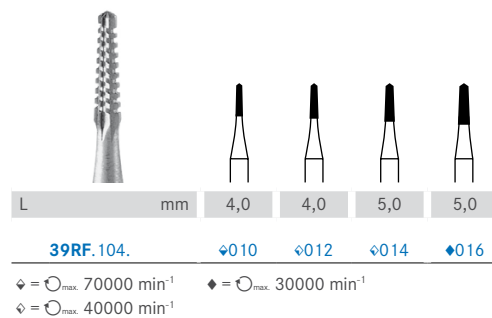*With cross-cut, for the treatment of the nail fold*

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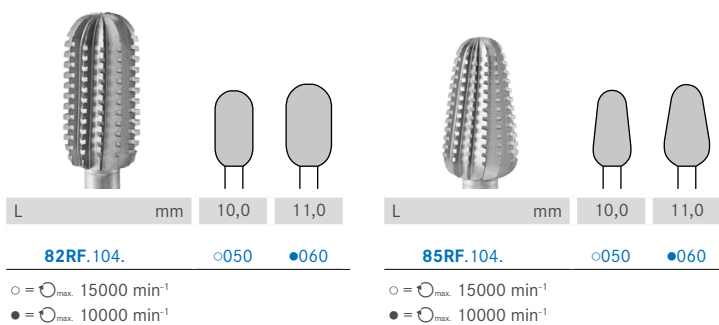
*Coarse special cutters*

*With cross-cut, for the universal treatment of hard skin*

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
*Coarse special cutters*

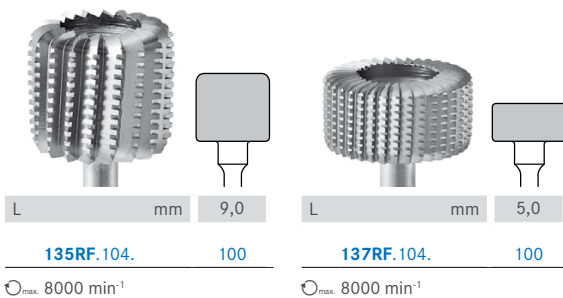*With cross-cut, for the removal of very hard skin and calluses*

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
*Special cutter*

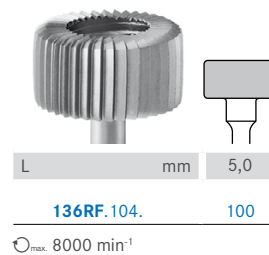*Without cross-cut, for the reduction of thickened nails*

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
#### Hollow cutter (trepan)

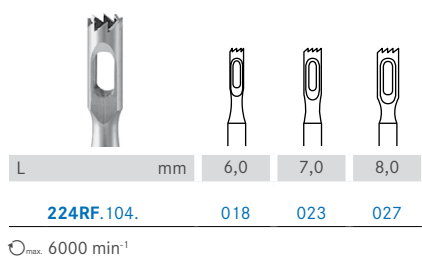With toothed blade, for the removal of calluses and cracked, hard skin

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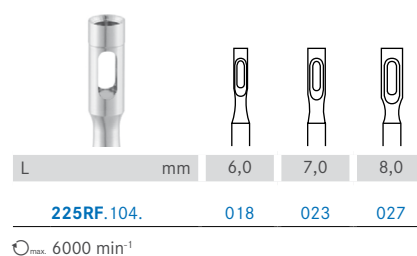
#### Rotary tissue punch

With ground blade, for the removal of deep-set corns

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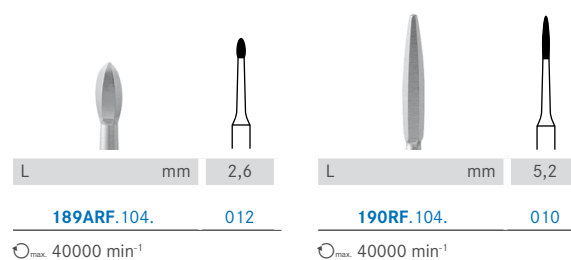
#### Nail cleaners


Hexagonal working part (without sharp blades) for gently removing deposits from the nail fold and cuticles

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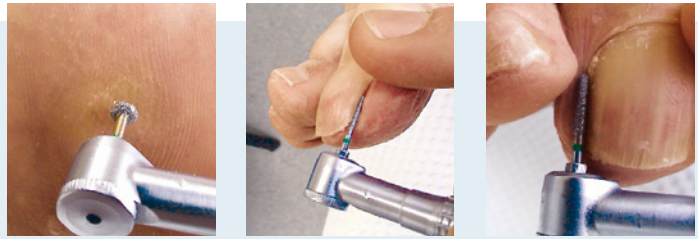#### *Anthracite polishers*

*For the smoothing and pre-polishing of natural and artificial nails*

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#### *Khaki polishers*

*For polishing natural and artificial nails*

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#### Microfiber mop

For the high-shine polishing of natural and artificial nails

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#### Goat hair brush

For cleaning the nail after the treatment

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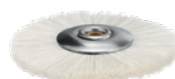
## General instructions for use and safety recommendations for the application of rotary pedicure instruments

### Scope

- These general instructions for use and safety recommendations apply to all products and are to be generally observed.
- Separate instructions for use are enclosed in the packaging of products that require more detailed information. These take precedence over the general instructions

### Proper use

- Make sure that only technically and hygienically perfect and cleaned turbines and hand pieces are used.
- Chuck the instruments as deeply as possible.
- The instrument must be rotating at the desired speed before contact is made with the object.
- Avoid jamming and using the instrument as a lever as this leads to an increased risk of breakage.
- Due to the generation of dust during use, we recommend working with an aspirator and spray. Wear a protective face mask.
- Avoid unprotected contact with the instruments (wear disposable protective gloves).
- To prevent thermal damage caused by rotary instruments, work at low speed and/or provide sufficient cooling.
- To avoid excessive heat generation, work with low pressure.
- Improper use leads to increased risk and inferior results. Therefore, stick to the recommended speed indicated on the labels and in the instructions for use.

### Recommended speed

- The general rule is: The larger the working part, the slower the speed
- A recommended maximum speed of  $\text{opt. } 6,000 - 100,000 \text{ rpm}$  means: Suited for micro motor hand-pieces up to the indicated speed
- Working parts with diameters that exceed that of the shank can cause strong centrifugal forces, when used at high speeds, resulting in the deformation of the shank or fracture of the instrument. Therefore, do not exceed the permitted maximum speed.
- The maximum permitted speed is indicated on the packaging.

### Disposal of worn instruments

- Damaged and deformed blades cause vibrations and lead to rough surfaces and increased heat generation.
- Void spots on the surface of diamond instruments can be caused by missing diamond grit and indicate that the instrument is blunt. The use of blunt instruments leads to excessive temperatures and can cause damage to the tissue.
- Deformed and/or non-concentric instruments must be discarded immediately.

### Storage and reprocessing

- Before first use, instruments should be stored in their original packages at room temperature protected against dust and humidity.
- All Komet instruments and parts are wrapped in non-sterile conditions. Before first use on the patient and immediately after each use, they have to be disinfected, cleaned and – if necessary and material-compatible – sterilized. Improper cleaning and sterilisation of the instruments can lead to the patient being infected with hazardous germs.

### Recommended reprocessing procedure:

1. Completely immerse instruments in a disinfection solution. Use suitable anti-corrosive disinfecting and cleaning agents at all times. For recommendations of use (immersion time, concentration, suitability) see instructions provided by the manufacturer of these agents.
2. Remove instruments from the disinfection solution, rinse thoroughly with running water and dry. Remove stubborn residues with a nylon brush, turning the instrument constantly, and disinfect again. Repeat the cleaning and disinfection process until all visible contamination has been removed. Pay special attention to the hollow spaces, if applicable. Note: The use of an ultra sonic device accelerates the disinfection and

improves the cleaning effect. The instruments must not come into contact with one another during the treatment in the ultrasonic device. Observe the instructions for use provided by the device manufacturer.

3. This is followed by a visual inspection of the cleaned and dried instruments. Do not reuse any damaged or blunt instruments which must be discarded.
  4. A final sterilisation cycle is recommended. All instruments intended for sterile use have to be sealed in suitable foil prior to sterilisation in the autoclave. Residues of detergents and lime scale can lead to discoloration or corrosion during sterilisation. Do not reuse corroded instruments
- The reprocessed instruments must be stored in hygienic containers, such as bur blocks or bowls that have been reprocessed according to the same procedure. Plastic stands from sales packages are unsuitable for the storage of reprocessed instruments.
  - The same applies to sterilised instruments and instruments in sterile goods packages.
  - Store away from dust, moisture and protected from possible sources of recontamination.
  - Protect polishers from UV rays during storage.

### Material compatibility

- As a general rule, all disinfection and cleaning agents used should be anti-corrosive.
- Polishers and high-grade corundum abrasives should be reprocessed with suitable, non-alcoholic agents.
- Bur blocks and boxes made of aluminium might be attacked and damaged by chemicals. Make sure to use only suitable cleaning and disinfection agents.
- Komet instruments are suitable for sterilisation in the autoclave at 134 °C and hot air sterilisation at 180 °C, with the exception of polishers and high-grade corundum abrasives which may only be sterilised in the autoclave at 134 °C. All polishers are unsuitable for sterilisation!
- The end of the service life of each Komet instrument is determined by its extent of wear and damage. Frequent reprocessing does not impair the performance of these products.

### Liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be discarded and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure have to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage to the hand piece or injury. The user is responsible for checking the product prior to use to ensure that it is suitable for the intended purpose.

In cases of contributory negligence by the user, Komet partly or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

## Explanation of the pictograms on the packaging labels

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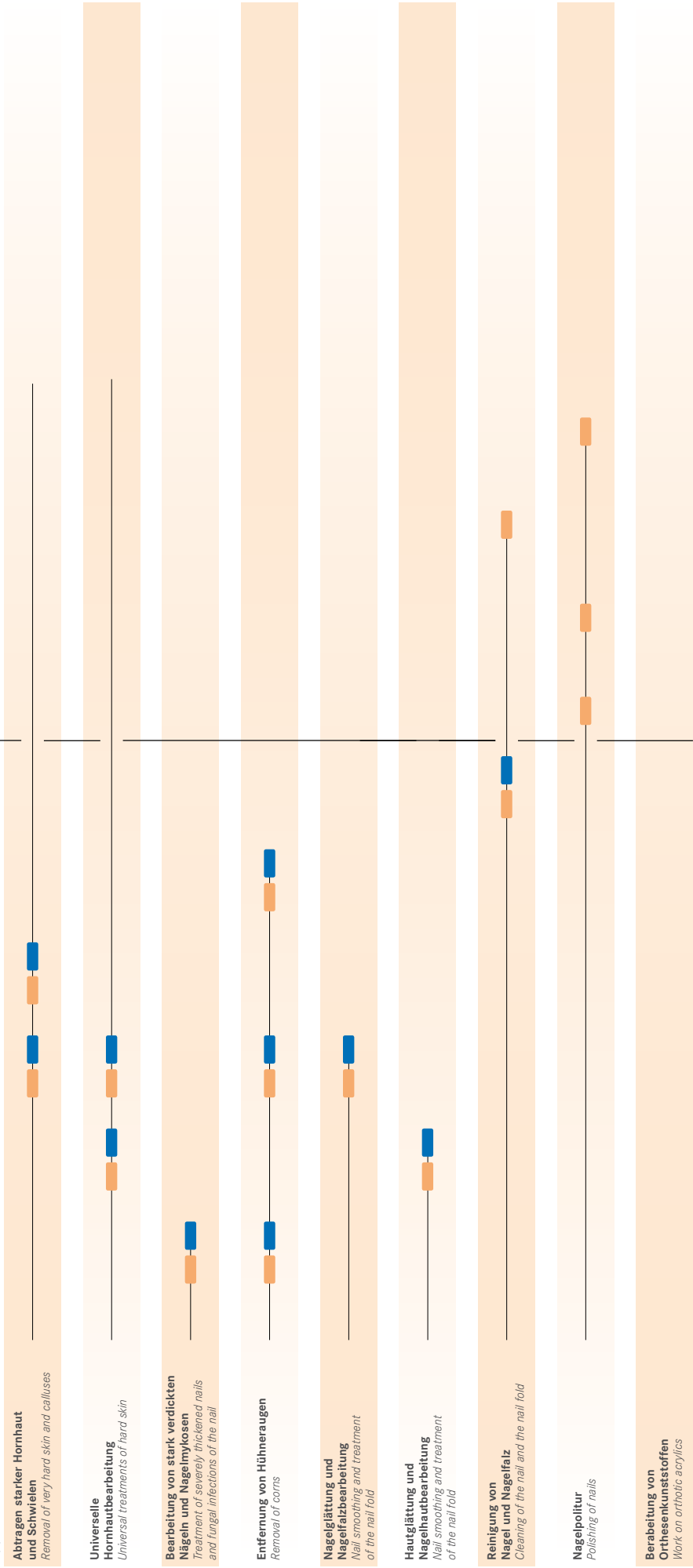
A series of horizontal dotted lines spanning the width of the page, providing a template for handwriting practice.

A series of horizontal dotted lines spanning the width of the page, intended for writing or drawing.